

1

Install the Tabletop (On Motorized / Manual Table)

Remove the original table top from the motorized table or manual table.



Remove the tabletop from the box.



Remove the mounting post from the bottom of the slit lamp table top.




Mount the slit lamp table top directly to the motorized table or manual table by using phillips head wood screws. (Recommended screw size: #10, Pan Head, 1")

2

Install the Headrest Assembly



Feed headrest wires through mounting holes on the table top.

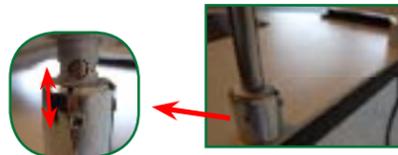

Insert the headrest posts through the mounting holes and ensure the guide marks are aligned. Then tighten the set screws on each mounting holes with the hex key provided.



3

Install the Slit Lamp Body



Remove the slit lamp body from the box.



Place the slit lamp body onto the table top with the wheels on the track.



Install track covers.



Pull the eye cap out from its shipping position.



4

Install the Accessories & Connect the Wires

4-1



Install chinrest paper and position with the chinrest pins.

4-2



Install patient handles on each side of the headrest.

4-3



Install the breath shield on the back of the optical head.

4-4



Connect the light intensity control cable from the power supply box to the top of the slit lamp base.

4-5



Connect the main power cable, fixation lamp cable, lamp output cable, and light intensity control cable to the back of the power supply.

CONGRATULATIONS!
YOU JUST COMPLETED
THE INSTALLATION...

Plug In, Turn On and enjoy your iBEX® Slit Lamp!
Just let us know of any questions at www.iBEXeye.com. Thank you!